

Instructions for use

Swab

Intended Use:

Intended to be used to apply medications to, or to take specimens from, a patient.

Intended User:

Intended to be used by Healthcare Professionals on patients that require application of medications or specimen collection.

Precautions:

- This device is single use only. Reuse of this device may lead to cross contamination and/or device failure.
- Do not re-sterilize.
- Do not use if product or package is damaged.
- Do not use for purposes other than intended use.
- During use, avoid excessive force or strain on the swab in order to preserve device integrity.
- Do not use if bud end of swab contacts anything other than intended medication/specimen site.
- Specimens collected from a patient or areas where medication is applied can contain infectious substances. Seek proper training and instruction on facility protocols before applying medication or collecting/ handling specimen.
- The use of this swab for any specific medication application or testing procedures should be validated by the user.
- Device is in intended for limited contact duration, less than 24 hours.

Instructions for Use:

1. Don appropriate personal protective equipment per facility protocols.
2. Peel open sterile pouch.
3. Remove swab from package, without touching bud end of swab.
4. Use swab to apply medication or collect specimen from patient. Ensure bud end only touches area of medication application or specimen collection.
5. If applicable, process the swab following the procedure required for the specimen being collected.
6. Dispose of contaminated devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.



Aspen Surgical Products, Inc.

6945 Southbelt Dr. SE, Caledonia, MI 49316

phone +1 616.698.7100 toll-free +1 888.364.7004

aspensurgical.com

Made in USA



STERILE EO

Rx ONLY

1050051 rev 1