# Instructions for Use

## Bard-Parker® Conventional Blade System

### Non-Sterile

### Intended Use

Intended for tissue separation and other procedures that require a sharp surgical blade to puncture or cut.

### Precautions

- The way in which a surgical blade is handled prior to use can affect the way in which the blade performs during use. Take care to ensure the cutting edge of a blade does not become damaged once removed from its packaging. This includes dropping the blade into a metal bowl or other container that could potentially dull the blade. It also includes not gripping the blade with a forceps or needle clamp across the cutting edge of the blade.
- During use, avoid putting excessive force or strain on the blade in order to help prevent breakage.
- Keep in mind that surgical blades are sharp instruments. Take proper precautions in handling the blade so as not to injure yourself or others before, during, or after use. Seek proper training and instruction on use before handling.
- This device is single use only. Reuse of this device may lead to patient cross contamination and/or device failure.
- During use avoid twisting, bending or putting excessive force or strain on the blade in order to help prevent breakage.
- A singular blade is intended to cut tissue up to 24 inches.
- If blade becomes dull or breaks, dispose and replace product.

### Sterilization Instructions

1. Place device into a kit or sterilization tray.
2. Using Ethylene Oxide sterilization method, sterilize according to the following parameters

Cycle Parameter	Specification	Duration
<b>Preconditioning</b>		
Temperature	110°F +/- 10°F	
Humidity	65% +/- 15%	10.35 Hrs -72 Hrs
<b>Chamber</b>		
Vacuum	21" +/- 1.0 inHgA	
Humidification/Dwell	22.5" +/- 1.0 HgA	60 +30/-0 minutes
Gas Inject/Dwell	10.3" +/- 0.5 Hg rise	360 +5/-0 minutes
Jacket Temperature	135°F	
<b>Aeration</b>		
Temperature	120°F	24 Hrs – 72 Hrs Max

### Instructions for Use

#### Attaching the Surgical Blade

1. Peel pouch open. Grip blade with forceps or needle clamp and remove from pouch, taking special care not to grip across the cutting edge of the blade itself.
2. Holding on to the thickest part of blade with forceps or needle clamp, insert the handle into blade track.
3. Slide blade back onto handle until it clicks into position.

#### Removing the Surgical Blade

1. Grip bottom of blade with forceps or needle clamp and pull up to loosen blade from handle.
2. Push blade forward from handle to remove blade, being careful not to jerk the blade off the track.
3. Dispose in an approved puncture-resistant sharps container according to facility protocol.

Does not contain natural rubber latex

**Rx ONLY** Caution: Federal law restricts this device to sale by or on the order of a physician

Aspen Surgical Products, Inc.  
6945 Southbelt Dr. SE  
Caledonia, MI 49316 USA  
Phone 616-698-7100  
Toll-Free 888-364-7004  
www.aspensurgical.com

EC REP

Hill-Rom Limited  
Clinitron House, Ashby Park  
Ashby De La Zouch  
Leicestershire, LE65 1JG  
England

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