INSTRUCTIONS FOR USE

Bard Parker®

aspensurgical.

Protected Blade System

Aspen Surgical Products, Inc.
6945 Southbelt Drive SE
Caledonia, MI 49316 USA
Phone 616-698-7100
Toll-Free 888-364-7004
aspensurgical.com

EC REP
Emergo Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands
Made in USA

CE
2797

8365946 | DGP65 | REV9 | 01DEC2020

English

Bard-Parker® Protected Blade System

Intended Use:

Intended for tissue separation and other procedures that require a sharp blade to puncture or cut.

Intended User/ Patient Target Groups:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Precautions:

- This device is single use only. Reuse of device could result in infection/contamination and/or device failure which could result in patient harm.
- Ruler shown on scalpel handle should be used as an indicator only, and not for accurate measurements.
- During use avoid twisting, bending or putting excessive force or strain on the blade in order to help prevent breakage.
- A singular blade is intended to cut tissue up to 24 inches.
- If blade becomes dull or breaks, dispose and replace product.
- If safety shield or housing malfunction, dispose and replace product.
- Do not use product if product is damaged or particulate is found in sterile packaging.

Assemble:

- Grasp protective shield with thumb and index finger with blade tip pointing up.
- Insert handle cleat and attach protected blade up to the indicator line.

CAUTION: DO NOT place hand in front of shield opening:

Use

- Depress top button on protective shield.
- Retract shield to rear locked position to fully uncover blade.

Cover

- Depress top button on protective shield.
- Advance shield to forward locked position at lock indicator line.

Dispose

One-handed technique with blade covered:

- Place thumb on side of handle behind cartridge.
- Push forward to unload protected blade.
- Dispose in approved puncture-resistant sharps container according to facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Notice to Users and/or Patients in EU:

Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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