# Instructions for Use

## Bard-Parker® Protected Blade System

### Intended use

Intended for tissue separation and other procedures that require a sharp surgical blade to puncture or cut.

### Precautions

- This device is single use only. Reuse of this device may lead to patient cross contamination and/or device failure.
- Ruler shown on scalpel handle should be used as an indicator only, and not for accurate measurements.
- During use avoid twisting, bending or putting excessive force or strain on the blade in order to help prevent breakage.
- A singular blade is intended to cut tissue up to 24 inches.
- If blade becomes dull or breaks, dispose and replace product.
- If safety shield or housing malfunction, dipose and replace product.

### Assemble

1. Grasp protective shield with thumb and index finger with blade tip pointing up.
2. Insert handle cleat and attach protected blade up to the indicator line.

**CAUTION: DO NOT place hand in front of shield opening.**

### Use

1. Depress top button on protective shield.
2. Retract shield to rear locked position to fully uncover blade.

### Cover

1. Depress top button on protective shield.
2. Advance shield to forward locked position at lock indicator line.

### Dispose

#### **One-handed technique with blade covered:**

1. Place thumb on side of handle behind cartridge.
2. Push forward to unload protected blade.
3. Dispose of protected blade in an approved sharps container in accordance with your facility's sharp disposal procedures.

Do not use if package is damaged

Does not contain natural rubber latex

**Rx ONLY**

Caution: Federal law restricts this device to sale by or on the order of a physician

Aspen Surgical Products, Inc.  
6945 Southbelt Dr. SE  
Caledonia, MI 49316 USA  
Phone 616-698-7100  
Toll-Free 888-364-7004  
www.aspensurgical.com

EC REP

Hill-Rom Limited  
Clinitron House, Ashby Park  
Ashby De La Zouch  
Leicestershire, LE65 1JG  
England

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