Instructions for Use

Bard-Parker® Protected Disposable Scalpel Important Usage Guidelines

Intended Use

Intended for tissue separation and other procedures that require a sharp surgical blade to puncture or cut.

Precautions

- This device is single use only. Reuse of this device may lead to patient cross contamination and/or device failure.
- Ruler shown on scalpel handle should be used as an indicator only, and not for accurate measurements.
- During use avoid twisting, bending or putting excessive force or strain on the blade in order to help prevent breakage.
- A singular blade is intended to cut tissue up to 24 inches.
- If blade becomes dull or breaks, dispose and replace product.
- If safety shield or housing malfunction, dipose and replace product.

To Use

1. Depress top button on protective shield.
2. Retract shield to rear locked position to fully uncover blade.

To Cover

1. Depress top button on protective shield.
2. Advance shield to forward locked position at lock indicator line.

Disposal

Dispose protected scalpel in an approved sharps container in accordance with your facility's sharps disposal procedures.

Do not use if package is damaged

Does not contain natural rubber latex

Rx ONLY

Caution: Federal law restricts this device to sale by or on the order of a physician

Aspen Surgical Products, Inc.
6945 Southbelt Dr. SE
Caledonia, MI 49316 USA
Phone 616-698-7100
Toll-Free 888-364-7004
www.aspensurgical.com

EC REP

Hill-Rom Limited
Clinitron House, Ashby Park
Ashby De La Zouch
Leicestershire, LE65 1JG
England

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