

Instructions for use

Carson 8" Perineal Post Pad 712200

Previously Beatty® reorder # 071801

Intended Use:

Intended to support and maintain the position of an anesthetized patient during surgery.

Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Precautions:

- For use with the Smith & Nephew® Supine Hip Positioning System 72200624 and Mizuho OSI®, Hana®, ProFx®, and Hana SSXT® Table.
- Ensure pad is secure in the positioning device and that it stays in place while the patient is on the table.
- Use on recommended tables only.
- Ensure pad is centered between patients' legs.
- This device is single use only.
- Do Not Sterilize.

Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Instructions for Use:

1. Perineal post pad for hip scope or supine hip positioning.
2. Insert pad into positioning device securely.
3. Discard after use.

Smith & Nephew® is a registered trademark of Smith & Nephew PLC (Smith & Nephew). Mizuho OSI®, HANA®, PROFX®, SSXT®, are registered trademark or trademarks of Mizuho OSI, Inc. ("Mizuho OSI"). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by Smith & Nephew®, or Mizuho OSI®.



Aspen Surgical Products, Inc.

6945 Southbelt Dr. SE, Caledonia, MI 49316

phone +1 616.698.7100 toll-free +1 888.364.7004

aspensurgical.com

Made in USA



Rx ONLY