INSTRUCTIONS FOR USE

Dr. Fog®

aspensurgical.

6g Anti-Fog Solution, Sponge

Aspen Surgical Products, Inc.
6945 Southbelt Drive SE
Caledonia, MI 49316 USA
Phone 616-698-7100
Toll-Free 888-364-7004
aspensurgical.com

EC REP
Emergo Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands

Made in USA

CE
2797

English

Intended Use

Intended as an anti-fog agent for endoscopic camera lenses.

Intended User/ Patient Target Groups:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Instructions for use:

Remove paper backing from sponge and place sponge onto sterile drape. Remove cap and apply 5 or 6 drops of Anti-Fog solution to sponge. Touch end of scope to sponge, blot dry (do not wipe dry) with sterile sponge. Apply one drop of Anti-Fog solution to eye piece of scope, blot (do not dry) with sterile sponge. Repeat as needed. Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations. Warning: Avoid contact with eyes and prolonged contact with skin.

Precaution:

Device is single use only. Reuse of device could result in infection/contamination and/or device failure which could lead to patient harm. A singular anti-fog device is intended to prevent fogging when used and reapplied every 15 minutes up to 1 hour. This device may burn when used in the presence of high intensity heat source or flammable gas. Do not use product if product is damaged or particulate is found in sterile packaging.

Notice to Users and/or Patients in EU:

Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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