## INSTRUCTIONS FOR USE

**Bard-Parker®**  
**Reusable Handles -**  
**Non-Sterile**

aspensurgical.

Aspen Surgical Products Inc.  
6945 Southbelt Drive SE  
Caledonia, MI 49316  
Phone 616-698-7100  
Toll-Free 888-364-7004  
aspensurgical.com

EC REP

Emergo Europe  
Prinsessegracht 20  
21514 AP The Hague  
The Netherlands

### English

#### Reusable Handles

##### Intended use:

Intended to be used with surgical blades for tissue separation and other procedures that require a sharp surgical blade to puncture or cut.

##### Intended User/ Patient Target Groups:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

##### Precaution:

- Ruler shown on scalpel handle should be used as an indicator only, and not for accurate measurements.
- During use avoid twisting, bending or putting excessive force or strain on the blade in order to help prevent blade breakage.
- A singular blade is intended to cut tissue up to 24 inches.
- Do not use if product appears visually damaged or degraded.

##### Cleaning Procedure – Hand Washing

- After use, carefully remove scalpel blade and discard it. Do not allow blood or debris to dry on the handle.
- Immerse the scalpel handle in an enzymatic detergent prepared according to manufacturer's instructions and soak for 10 minutes.
- Scrub handle with a soft bristled brush (agitate while scrubbing) paying special attention to the surface area where the blade was attached.
- Rinse with warm tap water (38-49° C) for 1 minute.
- Wipe with a clean, lint free cloth to dry device.
- When dry, inspect device under magnification for any visible debris. If debris is visualized on the device surface after cleaning, repeat cleaning to remove all soils.

##### Cleaning Procedure – Washer/Disinfectant

- After use, carefully remove scalpel blade and discard it. Do not allow blood or debris to dry on the handle.
- Rinse the handle in cool water to remove any visible soil.
- Place items in the basket of the washer/disinfectant, handle side down.
- Run a typical cycle including a dry phase. Cycle used consisted of:
  - Prewash, 2' cold water
  - Enzyme wash, 4' hot tap water with Enzol (1 oz/gal)
  - Wash 1, 2' 65.6° with Valsure (0.25 oz/gal)
  - Rinse 1, 0:15 with hot tap water
  - Dry, 6' 98.8°
- Remove handles and inspect them under magnification for any visible debris.

##### Sterilization Procedure

Place handles directly into a peel pouch. Do not wrap the handles.

Using a Pre-Vac Autoclave, sterilize using the following parameters:

Cycle Parameter	Specification
Cycle Type	Pre-Vac
Sterilization Temperature	≥132° C
Sterilization Time	4 minutes
Dry Time	20 minutes

##### Disposal:

Dispose in approved puncture-resistant sharps container according to facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

##### Notice to Users and/or Patients in EU:

Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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