Sterion®

aspensurgical.

Suture Aid Booties

STERILE EO MD R_x ONLY

EC REP

Emergo Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands

Made in USA

English

Intended use:

Suture Aid Booties are intended to protect , tag and help locate sutures

Intended User/ Patient Target Groups:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Instructions for use

1. After removing backing from cartridge, apply to any appropriate surface.
2. Insert tip of appropriate instrument into Suture Aid Bootie and remove from cartridge.

Extra Suture Aid Bootie

- Apply Extra Aid Booties over tip of appropriate instrument.

Precaution:

Device is single use only. Reuse of device could result in infection/ contamination and/ or device failure which could lead to patient harm. Device is intended for limited contact duration, less than or equal to 8 hours. Do not use product if product is damaged or particulate is found in sterile packaging.

Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Notice to Users and/or Patients in EU:

Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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