INSTRUCTIONS FOR USE

Bard Parker®

SafeSwitch® Disposable

Scalpel Handle, Size 3 - (REF)ST-830DNS)

aspen surgical.

Aspen Surgical Products, Inc.
6945 Southbelt Drive SE
Caledonia, MI 49316 USA
Phone 616-698-7100
Toll-Free 888-364-7004
aspensurgical.com

EC REP
Emergo Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands

Made in USA

CE
2797

English

Bard-Parker® SafeSwitch® Disposable Scalpel Handle Cover

Intended Use

Intended to be used with surgical blades for tissue separation and other procedures that require a sharp surgical blade to puncture or cut.

- Use with blade sizes 10, 10A, 11, 15 and 15C only.

Intended User/ Patient Target Groups:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Precautions

- Verify that the shield locks on both front and rear locking positions before use. Do not use scalpel handle if failure is observed. Do not use scalpel handle if signs of breakage is observed.
- Do not remove scalpel handle from cover until the blade is removed from the handle.
- Ruler shown on scalpel handle should be used as an indicator only, and not for accurate measurements.
- During use avoid twisting, bending or putting excessive force or strain on the blade in order to help prevent breakage.
- This device is single use only. Reuse of device could result in infection/contamination and/ or device failure which could lead to patient harm.
- Conventional Handle must be inserted and removed from when no blade is attached.
- A singular blade is intended to cut tissue up to 24 inches.
- Do not use if product appears visually damaged or degraded.

Sterilization Instructions

1. Place device into a kit or sterilization tray.
2. Using Ethylene Oxide sterilization method, sterilize according to the following parameters:

Cycle Parameter	Specification	Duration
Preconditioning		
Temperature	110°F +/- 10°F	
Humidity	65% +/- 15%	10.35 Hrs - 72 Hrs
Chamber		
Vacuum	21" +/- 1.0 inHgA	
Humidification/Dwell	22.5" +/- 1.0 HgA	60 +30/-0 minutes
Gas Inject/Dwell	10.3" +/- 0.5 Hg rise	360 +5/-0 minutes
Jacket Temperature	135°F	
Aeration		
Temperature	120°F	24 Hrs - 72 Hrs Max

Instructions for Use:

Conventional Handle Insertion

Ensure that the shield is in the forward locked position. Insert the tip of the conventional handle into the open slot on the back of the cover. Slide the conventional handle into the cover. Ensure that the tip of the handle goes under the shield. Push the handle until it fits completely into the cover.

Blade Assembly

Load blade following standard technique for left or right handed users.

Scalpel Use

To move the shield, press side locking button and move the shield forward to cover the blade. Press the same button and move the shield backward to rear locked position to use the blade. It is not required to hold the button down while sliding the shield after it has been unlocked.

Blade Removal

Move shield forward so the shield covers the blade. Orient handle downward towards a proper disposal location. Press side locking button and continue to move shield forward until blade is released from the handle. Pull shield back to front locked position after blade is removed. The blade will automatically drop from the shield.

Conventional Handle Removal

Ensure that the shield is in the forward locked position and there is no blade on the handle. Push the conventional handle out of the cover at the circular opening on the front of the cover. Pull the conventional handle down to remove it from the cover.

Disposal:

Dispose in approved puncture-resistant sharps container according to facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Notice to Users and/or Patients in EU:

Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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