Instructions for Use

WriteSite® Plus Skin Markers

Indications for Use

- Skin marker is used for surgical marking on intact skin.

Precautions

- Formulated specifically for ChloraPrep® products. Also works well with other popular prep solutions.
- The skin marker ink must be applied to intact, dry skin before applying prep solution. The skin marker ink will not adhere to prep solution or wet skin.
- Prepping the skin with isopropyl alcohol before marking may increase the adherence of the ink to the skin.
- When prep solution is applied over the skin marker ink, some of the marking will smear, but the skin marker ink will still be visible.
- This device is single use only. Reuse of this device may lead to patient cross contamination and/or device failure.
- Rulers packaged with skin markers are not measuring devices and should be used as an indicator only, and not for accurate measurements.
- Not for ocular use.
- Not for use on infants.
- Device is intended for limited contact duration, less than or equal to 24 hours.

Instructions for Use

1. Remove sterile pen from packaging.
2. Remove the pen cap to expose the pen tip.
3. Mark desired areas on skin with the skin marker according to facility protocol.
4. Allow the markings to dry on the skin for approximately 1 minute before rubbing or applying prep solution.
5. Apply prep solution as instructed by prep solution manufacturer.

Aspen Surgical Products, Inc.
6945 Southbelt Dr. SE
Caledonia, MI 49316 USA
Phone 616-698-7100
Toll-Free 888-364-7004
www.aspensurgical.com

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