

Instructions for use

Dr. Don's® Arm Canoe 711500

Previously Beatty® reorder # 020801

Intended Use:

Intended to support and maintain the position of an anesthetized patient during surgery.

Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Precautions:

- For use with the ARTHREX® Lateral Decubitus Shoulder Traction Tower and ARTHREX® 3-Point Shoulder Distraction System to achieve lateral position distraction allowing joint access.
- When hooking to the positioning device ensure weight is maintained on the device to avoid the arm from dropping.
- Ensure wrap is properly secured to the arm to avoid possible slippage and contamination of the sterile field.
- This device is single use only.
- Do not Sterilize.
- Do not use product if product is damaged or particulate is found in sterile packaging.

Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Instructions for Use:

1. Place arm in foam holder after it has been aseptically prepped for surgery.
2. Wrap with self-adhering wrap tightly.
3. Hook to the distraction device and apply the manufacture's recommended weight to the device for optimum shoulder distraction.
4. Discard after use.

Arthrex® is a registered trademarks of Arthrex®, Inc. ("Arthrex®"). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by Arthrex®.



Aspen Surgical Products, Inc.

6945 Southbelt Dr. SE, Caledonia, MI 49316

phone +1 616.698.7100 toll-free +1 888.364.7004

aspensurgical.com

Made in USA



STERILE | R

Rx ONLY

1047546 rev 1