

Instructions for use

Dr. Don's® Arm Canoe - V 711501

Previously Beatty® reorder # 020801-V

Intended Use:

Intended to support and maintain the position of an anesthetized patient during surgery.

Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Precautions:

- For use with the ARTHREX® Lateral Decubitus Shoulder Traction Tower, ARTHREX® 3-Point Shoulder Distraction System.
- When hooking the canoe to the positioning device ensure weight is maintained on the device to avoid the arm from dropping.
- Ensure wrap is properly secured to the arm to avoid possible slippage and contamination of the sterile field.
- This device is single use only.
- Do not Sterilize.
- Do not use product if product is damaged or particulate is found in sterile packaging.

Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Instructions for Use:

1. Place arm in foam holder after it has been aseptically prepped for surgery.
2. Wrap with self-adhering wrap tightly.
3. Hook to the distraction device and apply the manufacture's recommended weight to the device for optimum shoulder distraction.
4. Discard after use.

Arthrex® is a registered trademark of Arthrex, Inc. ("Arthrex"). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by Arthrex Inc.



Aspen Surgical Products, Inc.

6945 Southbelt Dr. SE, Caledonia, MI 49316

phone +1 616.698.7100 toll-free +1 888.364.7004

aspensurgical.com

Made in USA



STERILE | R

R_x ONLY