

## Instructions for use

### Dr. Don's® Arm Suspension Kit - Northern 711504

Previously Beatty® reorder # 092201

#### Intended Use:

Intended to support and maintain the position of an anesthetized patient during surgery.

#### Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

#### Precautions:

- When hooking to the positioning device ensure weight is maintained on the device to avoid the arm from dropping.
- Ensure wrap is properly secured to the arm to avoid possible slippage and contamination of the sterile field.
- This device is single use only.
- Do not re-sterilize.
- Do not use if product is damaged or particulate is found in sterile packaging.

#### Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

#### Instructions for Use:

1. Present sterile wrapped goods to sterile field following appropriate sterile technique.
2. Unwrap sterile CSR wrap.
3. Place arm in sleeve after it has been aseptically prepped for surgery.
4. Wrap with self-adhering wrap tightly.
5. Hook to the distraction device and apply the manufacturer's recommended weight to the device for optimum shoulder distraction.
6. Discard after use.



Aspen Surgical Products, Inc.

6945 Southbelt Dr. SE, Caledonia, MI 49316

phone +1 616.698.7100 toll-free +1 888.364.7004

[aspensurgical.com](http://aspensurgical.com)

Made in USA



STERILE R

**Rx ONLY**