

Instructions for use

Erin Beach Chair Face Mask 711001, Non-Sterile

Previously Beatty reorder # 021501

Intended Use:


Intended to support and maintain the position of an anesthetized patient during surgery.

Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Precautions:

- For use with Smith & Nephew® T-MAX™ Beach Chair 7210996.
- Ensure mask is properly aligned over eyes and nose as shown in drawing.
- Ensure that skin & hair are away from straps when securing device.
- Ensure that the straps are securely fastened to the positioning device.
- Do not overtighten straps on mask.
- This device is single use only.
- Do not sterilize.

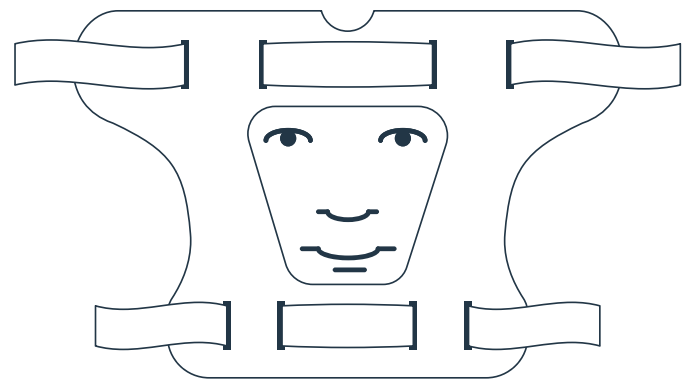

Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Instructions for Use:

1. Place mask over patient's face as shown in drawing.
2. Secure buckles into receiving clips on positioning device.
3. Tighten straps if necessary (see precaution above).
4. Discard after use.

Smith & Nephew T-MAX is a registered trademark of Smith & Nephew PLC (Smith & Nephew). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by Smith & Nephew.



Aspen Surgical Products, Inc.

6945 Southbelt Dr. SE, Caledonia, MI 49316

phone +1 616.698.7100 toll-free +1 888.364.7004

aspensurgical.com

Made in USA



Rx ONLY

1046420 rev 1