

## Instructions for use

### Hermanator Knee Positioning Pad 713100

Previously Beatty® reorder # 112001

#### Intended Use:

Intended to support and maintain the position of an anesthetized patient during surgery.

#### Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

#### Precautions:

- Protective insert for the foot holder for: IMP® De Mayo Knee Positioner® 803 AND 803-S, Zimmer® Alvarado Knee Holder 1320-000-00, SchureMed™ TKR Knee Positioner 800-0141\_800-0171, and Innomed® Robb Knee Positioner 2630-11.
- Ensure foam forms to patient's foot and leg to protect against mechanical skin injury.
- Wrap foot securely with self-adhering wrap to avoid slipping out of the positioning device.
- This device is single use only.
- Do not re-sterilize.
- Do not use product if product is damaged or particulate is found in sterile packaging.

#### Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

#### Instructions for Use:

1. Form the foam foot piece around patient foot and calf.
2. Place foot in the positioning device.
3. Wrap foam foot piece and foot with self-adhering wrap around the positioning device.
4. Discard after use.

IMP® DEMAYO KNEE POSITIONER is a registered trademark of Innovative Medical Products, Inc. ("IMP"). Zimmer® Alvarado Knee Holder is a registered trademark of Zimmer, Inc. ("Zimmer"). SchureMed™ is a trademark of Schuerch Corporation ("Schuerch"). Innomed® is a registered trademark of Innomed, Inc. ("Innomed"). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by IMP, Zimmer, Schuerch, or Innomed.



Aspen Surgical Products, Inc.

6945 Southbelt Dr. SE, Caledonia, MI 49316  
phone +1 616.698.7100 toll-free +1 888.364.7004  
[aspensurgical.com](http://aspensurgical.com)

Made in USA



STERILE | R

**R<sub>x</sub> ONLY**