

Instructions for use

Hoffee® Blue Lasso Suture Retriever, Non Sterile

Intended Use:

Intended to aid in the passing of sutures through bone and soft tissue during orthopedic surgery.

Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Precautions:

- When using arthroscopically, visualize the device as it is placed into the joint to avoid unintentional cartilage damage.
- Do not bend the device.
- Do not alter the suture on the device in any way.
- This device is single use only.
- Use caution when opening sterile package, device may roll.

Sterilization Instructions

- Place device into a kit or sterilization tray.
- Using Ethylene Oxide sterilization method, sterilize according to the following parameters:

Cycle Parameter	Specification	Duration
EC Gas Mixture	20% EO, 80% CO ₂	
Preconditioning		
Temperature	100°F +/- 20°F	
Humidity	55% +/- 15%	24 Hrs - 48 Hrs
Chamber		
Vacuum	18" +/- 0.5 inHgA	
Humidification/Dwell	16" +/- 0.5 inHgA	495 mins +/-15 minutes
Gas Inject/Dwell	12.5" +/- 0.5 Psig	870 mins +/-30 minutes
Jacket Temperature	130°F +/- 5°F	
Aeration		
Temperature	100°F +/- 20°F	72 Hrs Minimum

Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Instructions for Use:

1. Used to retrieve suture (surgical equivalent to a needle threader).
2. Place suture through the blue suture loop and pull through joint, bone or soft tissue.
3. Remove device from joint in a smooth motion.
4. Discard after use.