

## Instructions for use

### Indy Beach Chair Face Mask 711003

Previously Beatty® reorder # 100901

#### Intended Use:

Intended to support and maintain the position of an anesthetized patient during surgery.

#### Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

#### Precautions:

- For use with ALLEN® FLEXIFORM™ Head Rest A-90017.
- Ensure device is properly aligned above eyes and chin.
- Ensure that skin & hair are away from positioning device when securing device.
- This device is single use only.
- Do not sterilize.

#### Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

#### Instructions for Use:

1. Place around face and wrap over head.
2. Secure to positioning device.
3. Discard after use.

Allen® and Flexiform™ are registered trademark or trademarks of Allen Medical Systems, Inc. ("Allen Medical"). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by Allen Medical.



Aspen Surgical Products, Inc.

6945 Southbelt Dr. SE, Caledonia, MI 49316

phone +1 616.698.7100 toll-free +1 888.364.7004

[aspensurgical.com](http://aspensurgical.com)

Made in USA



**Rx ONLY**