

## Instructions for use

### Jessie Arm Suspension Kit 711300

Previously Beatty® reorder # 100401

#### Intended Use:

Intended to support and maintain the position of an anesthetized patient during surgery.

#### Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

#### Precautions:

- For use with McConnell Limb Positioner.
- Ensure wrap is securely attached to the positioning device while positioning arm.
- Ensure wrap is properly secured to the arm to avoid possible contamination of the sterile field.
- This device is single use only.
- Do not re-sterilize.
- Do not use if product is damaged or particulate is found in sterile packaging.

#### Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

#### Instructions for Use:

1. Present sterile wrapped goods to sterile field following appropriate sterile technique.
2. Unwrap sterile CSR wrap.
3. Create a fist with the fingers and place the hand in the small foam hexapod after it has been aseptically prepped for surgery.
4. Secure the foam fist wrap with the attached closure.
5. Place the larger foam piece around the arm and secure foam around arm.
6. Place foam arm holder securely in the positioning device.
7. Discard after use.

McConnell Orthopedic® is a registered trademark of McConnell Orthopedic® Manufacturing Company ("McConnell"). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by McConnell.



Aspen Surgical Products, Inc.

6945 Southbelt Dr. SE, Caledonia, MI 49316

phone +1 616.698.7100 toll-free +1 888.364.7004

[aspensurgical.com](http://aspensurgical.com)

Made in USA



STERILE R

**Rx ONLY**

1047547 rev 1