

Instructions for use

Lateral Positioner Pad 712600

Previously Beatty® reorder # 101400

Intended Use:

Intended to support and maintain the position of an anesthetized patient during surgery.

Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Precautions:

- For use with IMP® UNIVERSAL LATERAL POSITIONER® 1092.
- Take care when positioning patient to avoid any pinching or pressure on the skin.
- Single use only.
- Do not sterilize.

Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Instructions for Use:

1. Attach pad set to the base plate, back plate and support column.
2. Place patient in the lateral position.
3. Secure patient in place, following device manufacturers instructions.
4. Discard after use.

IMP® and Universal Lateral Positioner® 1092 are registered trademarks of Innovative Medical Products, Inc. ("IMP"). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by IMP.



Aspen Surgical Products, Inc.

6945 Southbelt Dr. SE, Caledonia, MI 49316

phone +1 616.698.7100 toll-free +1 888.364.7004

aspensurgical.com

Made in USA



Rx ONLY