

## Instructions for use

### Leg Comfort Pad 713600

Previously Beatty® reorder # 062701

#### Intended Use:

Intended to support and maintain the position of an anesthetized patient during surgery.

#### Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

#### Precautions:

- For Use With Allen® Leg Holder R-734-33, Smith & Nephew® Acufex Knee Holder, SchureMed™ Schure Grip Leg Holder 800-0065, and Stryker® Arthroscopy Leg Holder OZ 7511975.
- Ensure pad is securely tightened around patient's leg.
- Be aware of skin to avoid pinching while placing patients' leg in positioning device.
- Do not place over a tourniquet.
- This device is single use only.
- Do not sterilize.

#### Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

#### Instructions for Use:

1. Place pad on upper thigh.
2. Place leg in positioning device ensuring it is secure.
3. Discard after use.

Allen® is a registered trademark of Allen Medical Systems, Inc. ("Allen Medical"). Smith & Nephew® is a registered trademark of Smith & Nephew PLC (Smith & Nephew). SchureMed™ is a registered trademark of Schuerch Corporation ("Schuerch"). Stryker® is a trademark of Stryker Corporation ("Stryker"). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by Allen Medical, Smith & Nephew, SchuerchMed, or Stryker.



Aspen Surgical Products, Inc.

6945 Southbelt Dr. SE, Caledonia, MI 49316  
phone +1 616.698.7100 toll-free +1 888.364.7004  
[aspensurgical.com](http://aspensurgical.com)

Made in USA



**Rx ONLY**