

Instructions for use

Lewiston Hip Positioner Set 712100

Previously Beatty® reorder # 072917

Intended Use:

Intended to support and maintain the position of an anesthetized patient during surgery.

Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Precautions:


- For use with Arthrex® Hip Distraction System AR-6529S.
- Ensure perineal pad is secure and covering the positioning device and that it stays in place while the patient is on the table.
- Ensure perineal pad is centered between patient's legs.
- Ensure boot liner is securely fastened around foot before placing in the positioning device.
- Avoid redundant boot material that may wrinkle or cause excess pressure.
- Use on recommended tables only.
- This device is single use only.
- Do not sterilize.

Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Instructions for Use:

1. Place perineal post pad onto positioning device securely.
2. Place patient's foot in liner.
3. Using the hook and loop closure, secure position 1 to 2, then 3 to 4 and lastly 5.
4. Secure foot in positioning device.
5. Secure boot to table using manufacturers recommended instructions.
6. Discard after use.

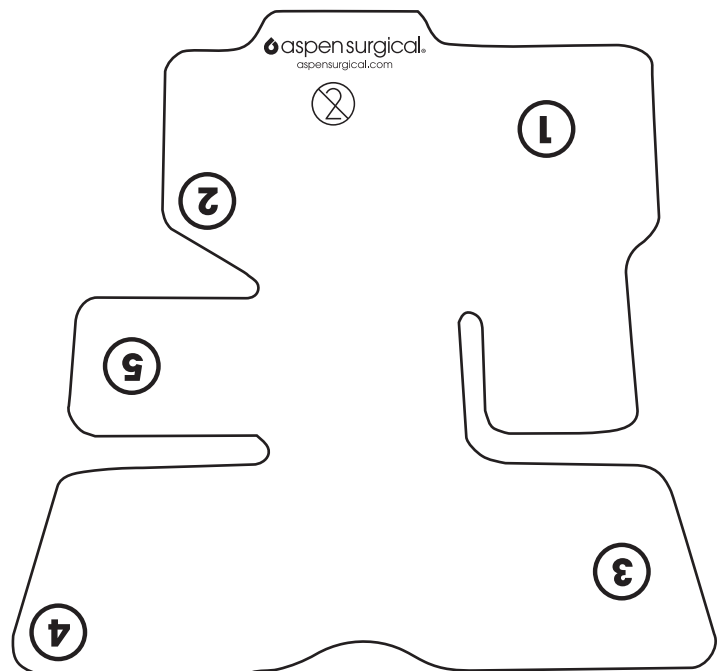

Arthrex® is a registered trademark of Arthrex, Inc. ("Arthrex"). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by Arthrex.

 Aspen Surgical Products, Inc.

6945 Southbelt Dr. SE, Caledonia, MI 49316
 phone +1 616.698.7100 toll-free +1 888.364.7004
aspensurgical.com

Made in USA



Rx ONLY

1047663 rev 1