

Instructions for use

Marco Shoulder Stabilization Kit 711100

Previously Beatty® reorder # 101601

Intended Use:

Intended to support and maintain the position of an anesthetized patient during surgery.

Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Precautions:

- SMITH & NEPHEW® SPIDER™ Limb Positioner 7210569.
- Ensure wrap is securely attached to the positioning device while positioning arm.
- Ensure wrap is properly secured to the arm to avoid possible contamination of the sterile field.
- Do not overtighten wrap and restrict blood flow.
- This device is single use only.
- Do not sterilize.
- Do not use product if product is damaged or particulate is found in sterile packaging.

Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Instructions for Use:

1. Place foam grip in patient hand before wrapping.
2. Wrap foam arm holder securely around arm after it has been aseptically prepped for surgery.
3. Place foam arm holder securely in the positioning device.
4. Discard after use.

Smith & Nephew, GUHL, T-MAX, SPIDER, SPIDER2, FERKEL, HEWSON SUTURE RETRIEVER, and SWANSON SUTURE PASSER are registered trademarks of Smith & Nephew PLC (Smith & Nephew).



Aspen Surgical Products, Inc.

6945 Southbelt Dr. SE, Caledonia, MI 49316

phone +1 616.698.7100 toll-free +1 888.364.7004

aspensurgical.com

Made in USA



STERILE | R

R_x ONLY

1047357 rev 1