

Instructions for use

MD Traction Rope 715000

Previously Beatty® reorder # 011987

Intended Use:

Intended to support and maintain the position of an anesthetized patient during surgery.

Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Precautions:

- For use with any suspension like device that utilizes rope.
- This device is single use only.
- Do not re-sterilize.
- Do not use if product is damaged or particulate is found in sterile packaging.

Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Instructions for Use:

1. Open package appropriately to maintain sterility of product.
2. Place rope in distraction device according to manufacturer's recommendations.
3. Hook to suspension kit and apply the manufacturer's recommended weight to the device for optimum shoulder distraction.
4. Discard after use.



Aspen Surgical Products, Inc.

6945 Southbelt Dr. SE, Caledonia, MI 49316

phone +1 616.698.7100 toll-free +1 888.364.7004

aspensurgical.com

Made in USA



STERILE R

Rx ONLY