

Instructions for use

Perineal Post Cover 712400

Previously Beatty® reorder # 082501

Intended Use:

Intended to support and maintain the position of an anesthetized patient during surgery.

Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Precautions:

- For use with Mizuho OSI®, Hana®, ProFx®, and Hana SSXT® Tables.
- Ensure post cover is secure on the positioning device and that it stays in place while the patient is on the table.
- Use on recommended tables only.
- Ensure pad is centered between patient's legs.
- This device is single use only.
- Do not sterilize.

Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Instructions for Use:

1. Insert over pad into positioning device securely.
2. Discard after use.

Mizuho OSI®, Hana®, ProFx® and Hana SSXT® are registered trademarks of Mizuho OSI, Inc. ("Mizuho OSI"). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by Mizuho OSI.



Aspen Surgical Products, Inc.

6945 Southbelt Dr. SE, Caledonia, MI 49316

phone +1 616.698.7100 toll-free +1 888.364.7004

aspensurgical.com

Made in USA



Rx ONLY