

## Instructions for use

### Racine Pelvic Positioner Pads 712800

Previously Beatty® reorder # 012802

#### Intended Use:

Intended to support and maintain the position of an anesthetized patient during surgery.

#### Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

#### Precautions:

- For use with IMP® MCGUIRE™ Hip Positioner 1084.
- Take care when positioning patient to avoid any pinching or pressure on the skin.
- Use with approved positioning device.
- Single use only.
- Do not sterilize.

#### Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

#### Instructions for Use:

1. Attach pad set to the base and back plate and support column.
2. Place patient in the lateral position.
3. Secure patient in place, following device manufacturers instructions.
4. Discard after use.

IMP®, Innovative Medical Products, and McGuire™ Hip Positioner 1084 are registered trademark or trademarks of Medical Products, Inc. ("IMP"). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by IMP.



Aspen Surgical Products, Inc.

6945 Southbelt Dr. SE, Caledonia, MI 49316

phone +1 616.698.7100 toll-free +1 888.364.7004

[aspensurgical.com](http://aspensurgical.com)

Made in USA



**Rx ONLY**