

## Instructions for use

### Smoot Ankle Distraction Strap 713400

Previously Beatty® reorder # 041501

#### Intended Use:

Intended to support and maintain the position of an anesthetized patient during surgery.

#### Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

#### Precautions:

- Strap designed to provide ankle distraction for the SMITH & NEPHEW® GUHL™ Positioner or the ARTHREX® Ankle Distractor.
- Ensure straps are equally distributed around posterior heel and anterior foot.
- Utilize sterile hook to avoid sterile field contamination.
- Use manufacture's recommended weight limits to avoid over distraction of the joint.
- This device is single use only.
- Do not sterilize.
- Do not use product if product is damaged or particulate is found in sterile packaging.

#### Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

#### Instructions for Use:

1. Place foot in straps, ensure heel is cradled in the labeled heel strap.
2. Apply appropriate distraction per device manufacture's recommendation.
3. Discard after use.

Smith & Nephew® and GUHL™, are registered trademarks of Smith & Nephew® PLC (Smith & Nephew®). Arthrex®, is a registered trademark of Arthrex®, Inc. ("Arthrex®"). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by Smith & Nephew® or by Arthrex®.



Aspen Surgical Products, Inc.

6945 Southbelt Dr. SE, Caledonia, MI 49316

phone +1 616.698.7100 toll-free +1 888.364.7004

[aspensurgical.com](http://aspensurgical.com)

Made in USA



STERILE | R

**R<sub>x</sub> ONLY**