

Instructions for use

TLC Distraction Pad Set 713200

Previously Beatty® reorder # 071617

Intended Use:

Intended to support and maintain the position of an anesthetized patient during surgery.

Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Precautions:

- For use with IMP® DEMAYO UNIVERSAL DISTRACTOR® 907.
- Ensure foam forms to patient's foot and leg to protect against mechanical skin injury.
- Ensure foam piece covers the entire edge of foot piece.
- Ensure the distractor pad is not covered by self adhesive wrap.
- Wrap foot securely with self-adhering wrap to avoid slipping out of the positioning device.
- This device is single use only.
- Do not re-sterilize.
- Do not use if product is damaged or particulate is found in sterile packaging.

Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Instructions for Use:

1. Present sterile wrapped goods to sterile field following appropriate sterile technique.
2. Unwrap sterile CSR wrap.
3. Form the foam foot piece around patient foot and calf.
4. Place foot in the positioning device.
5. Wrap foam foot piece and foot with self-adhering wrap around the positioning device.
6. Place foam distractor pad on device
7. Discard after use.

IMP®, Innovative Medical Products and De Mayo Universal Distractor® 907 are registered trademarks or trademarks of Innovative Medical Products, Inc. ("IMP"). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by IMP.



Aspen Surgical Products, Inc.

6945 Southbelt Dr. SE, Caledonia, MI 49316

phone +1 616.698.7100 toll-free +1 888.364.7004

aspensurgical.com

Made in USA



STERILE R

Rx ONLY

1047661 rev 1