

## Instructions for use

### Universal Head Restraint 711004

Previously Beatty® reorder # 050601

#### Intended Use:

Intended to support and maintain the position of an anesthetized patient during surgery.

#### Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

#### Precautions:

- For use with Allen® Beach Chair Positioner.
- Ensure device is properly aligned over eyes and chin.
- Ensure that skin & hair are away from straps when securing device.
- Ensure that the straps are securely fastened to the positioning device.
- Do not overtighten straps on mask.
- This device is single use only.
- Do not sterilize.
- Do not use product if product is damaged or particulate is found in sterile packaging.

#### Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

#### Instructions for Use:

1. Place foam around patient's face and chin securely.
2. Secure straps to positioning device.
3. Discard after use.

Allen® is a registered trademark of Allen Medical Systems, Inc. ("Allen Medical"). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by Allen Medical.



Aspen Surgical Products, Inc.

6945 Southbelt Dr. SE, Caledonia, MI 49316

phone +1 616.698.7100 toll-free +1 888.364.7004

[aspensurgical.com](http://aspensurgical.com)

Made in USA



**Rx ONLY**