## INSTRUCTIONS FOR USE

WriteSite® Plus

Multi-Ink Marking Pen

Non-Sterile

aspensurgical.

Aspen Surgical Products Inc.  
6945 Southbelt Drive SE  
Caledonia, MI 49316  
Phone 616-698-7100  
Toll-Free 888-364-7004  
aspensurgical.com  
Made in USA

EC REP

Emergo Europe  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

English

### SKIN MARKER

#### Intended Use:

Skin marker is used for surgical marking on intact skin.

#### Intended User/Patient Target Groups:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

#### Precaution:

- Works well with chlorhexidine gluconate (CHG) based prep solutions.
- The skin marker ink must be applied to intact, dry skin before applying prep solution. The skin marker ink will not adhere to prep solution or wet skin.
- Prepping the skin with isopropyl alcohol before marking may increase the adherence of the ink to the skin.
- When prep solution is applied over the skin marker ink, some of the marking will smear, but the skin marker ink will still be visible.
- This device is single use only. Reuse of device could result in infection/contamination and/or device failure which could lead to patient harm.
- Rulers packaged with skin markers are not measuring devices and should be used as an indicator only, and not for accurate measurements.
- Not for ocular use.
- Not for use on infants.
- Device is intended to write for up to 24 feet (7.32 meters).

#### Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

#### Instructions for Use:

1. Remove the pen cap to expose the pen tip.
2. Mark desired areas on skin with the skin marker according to facility protocol.
3. Allow the markings to dry on the skin for approximately 1 minute before rubbing or applying prep solution.
4. Apply prep solution as instructed by prep solution manufacturer.

### UTILITY MARKER

#### Intended Use:

Utility marker is used to provide a permanent method of identifying instruments, specimens, medication labels, and other objects on and off the sterile field.

#### Intended User/Patient Target Groups:

Intended to be used by Healthcare Professionals on instruments or objects in the operating room.

#### Precaution:

- Utility marker ink is not for use on skin.
- Contains a black, waterproof ink that can withstand autoclave sterilization on most surfaces.
- The device is single use only.
- Device is intended to write for up to 24 feet (7.32 meters).

#### Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

#### Instructions of Use:

1. Remove pen cap to expose pen tip.
2. Mark desired object or surface with the utility marker according to facility protocol.
3. Allow markings to dry on object for approximately 1 minute prior to use or autoclave sterilization.

#### Sterilization Instructions

To ensure SAL of 10<sup>-6</sup>, this product must be sterilized via gamma sterilization method in its original packaging configuration at a sterilization dose range of 28-50 kGy.

#### Notice to Users and/or Patients in EU:

Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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