

## Instructions for use

### Norton Arm Restraint Kit 711400

Previously Beatty® reorder # 031001

#### Intended Use:

Intended to support and maintain the position of an anesthetized patient during surgery.

#### Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

#### Precautions:

- For use with the ALLEN® arm positioner #A-92000.
- Ensure wrap is securely attached to the positioning device while positioning arm.
- Ensure wrap is properly secured to the arm to avoid possible contamination of the sterile field.
- This device is single use only.
- Do not re-sterilize.
- Do not use if product is damaged or particulate is found in sterile packaging.

#### Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

#### Instructions for Use:

1. Present sterile wrapped goods to sterile field following appropriate sterile technique.
2. Unwrap sterile CSR wrap.
3. Slide foam arm restraint over Allen® Arm Positioner.
4. Place arm on top of foam arm restraint.
5. Wrap the foam around the hand.
6. Secure foam using straps in a secure fashion.
7. Discard after use.

Allen® is a registered trademark of Allen Medical Systems, Inc. ("Allen Medical"). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by Allen.



Aspen Surgical Products, Inc.

6945 Southbelt Dr. SE, Caledonia, MI 49316

phone +1 616.698.7100 toll-free +1 888.364.7004

[aspensurgical.com](http://aspensurgical.com)



STERILE R

**Rx ONLY**