

Instructions for use

Hermantor XL Pink Knee Positioning Pad 713300

Previously Beatty® reorder # 080801

Intended Use:

Intended to support and maintain the position of an anesthetized patient during surgery.

Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Precautions:

- For use with IMP® De Mayo Knee Positioner® 803 and 803-S and Match Grade Ladderloc Knee positioner
- Ensure foam forms to patient's foot and leg to protect against mechanical skin injury.
- Wrap foot securely with self-adhering wrap to avoid slipping out of the positioning device.
- This device is single use only.
- Do not re-sterilize.
- Do not use if product is damaged or particulate is found in sterile packaging.

Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Instructions for Use:

1. Form the foam foot piece around patient foot and calf.
2. Place foot in the positioning device.
3. Wrap foam foot piece and foot with self-adhering wrap around the positioning device.
4. Discard after use.

IMP®, Innovative Medical Products, and De Mayo Knee Positioner® 803 and 803-S are registered trademarks of Innovative Medical Products, Inc. ("IMP"). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by IMP.



Aspen Surgical Products, Inc.

6945 Southbelt Dr. SE, Caledonia, MI 49316

phone +1 616.698.7100 toll-free +1 888.364.7004

aspensurgical.com



STERILE R

Rx ONLY