

Instructions for use

Bethesda Beach Chair Face Mask 71 1000, Non-Sterile

Previously Beatty reorder # 010601

Intended Use:


Intended to support and maintain the position of an anesthetized patient during surgery.

Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Precautions:

- For use with SchureMed™ Beach Chair 800-0004, Stryker® Shoulder Chair OZ7519475 and OZ 7523075, and Skytron® Beach Chair.
- Ensure mask is properly aligned over eyes and nose as shown in drawing.
- Ensure that skin & hair are away from straps when securing device.
- Ensure that the straps are securely fastened to the positioning device.
- Do not overtighten straps on mask.
- This device is single use only.
- Do not sterilize.

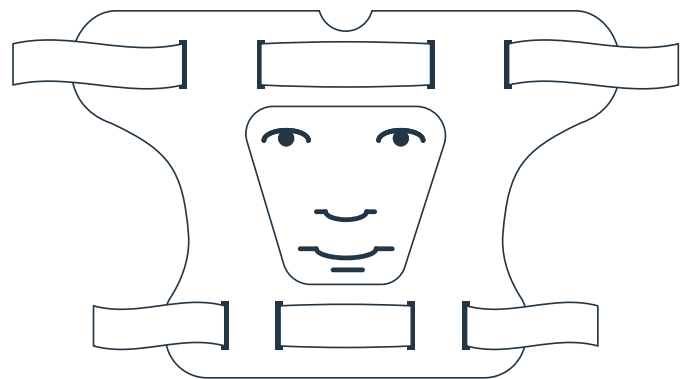

Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Instructions for Use:

1. Place mask over patient's face as shown in drawing.
2. Secure straps to positioning device.
3. Discard after use.

SchureMed is a trademark of Schuerch Corporation ("Schuerch"). Skytron is registered trademark of Skytron, LLC. ("Skytron"). Stryker is a trademark of Stryker Corporation ("Stryker"). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by Schuerch, Skytron, or Stryker.



Aspen Surgical Products, Inc.

6945 Southbelt Dr. SE, Caledonia, MI 49316
phone +1 616.698.7100 toll-free +1 888.364.7004
aspensurgical.com

Made in USA



Rx ONLY

1046419 rev 1