INSTRUCTIONS FOR USE

Richard-Allan®

aspensurgical.

Surgical Needles

STERILE EO

Rx ONLY MD

EC REP

Emergo Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands

Made in USA

CE
2797

Aspen Surgical Products, Inc.
6945 Southbelt Drive SE
Caledonia, MI 49316 USA
Phone 616-698-7100
Toll-Free 888-364-7004
aspensurgical.com

English

Richard-Allan® Surgical Needle Instructions for Use

Intended use:

Surgical Needles are Intended to be used with suture material for tissue closure.

Intended User/Patient Target Groups:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Instructions for use:

Improper surgical technique in the use of a suturing needle may result in needle breakage. Improper needle selection for the surgery intended may also result in needle breakage. Needle selection is the responsibility of the surgeon. Small needle holders should be used on fine gauge needles and heavy holders on heavy gauge needles to avoid needle breakage because of undue stress. The weakest parts of the needle are the eye and point. Do not clamp holder directly on the eye of the needle or grasp the point or breakage may occur. To avoid breakage, the needle holder (for example, the jaws of the forceps) should be in good condition and show no signs of deterioration and should be applied to the flat portion of the needle about 1/4 of a needle length from the eyed end. Do not use product if product is damaged or particulate is found in sterile packaging

Disposal:

Dispose in approved puncture-resistant sharps container according to facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Precaution:

Device is single use only. Reuse of device could result in infection/contamination and/or device failure which could lead to patient harm. A singular needle containing suture material is intended to puncture through tissue up to 30 times.

Notice to Users and/or Patients in EU:

Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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