INSTRUCTIONS FOR USE

Cautery Tip Cleaner 

Non-Sterile

Aspen Surgical Products Inc.
6945 Southbelt Drive SE
Caledonia, MI 49316
Phone 616-698-7100
Toll-Free 888-364-7004
aspensurgical.com

EC REP

Emergo Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands

Made in USA

English

Intended use:

Intended to assist in cleaning and removing material from cautery tips during surgical procedures.

Intended User/ Patient Target Groups:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Instructions for use

Remove backing and apply to appropriate surface. Wipe cautery tip on pad to remove debris.

Precaution:

Device is single use only. Reuse of device could result in infection/ contamination and/ or device failure which could lead to patient harm. Device is intended for limited contact duration, less than or equal to 16 hours.

Sterilization Instructions

- Place device into a kit or sterilization tray.
- Using Ethylene Oxide sterilization method, sterilize according to the following parameters:

| Cycle Parameter | Specification | Duration |
|------------------------|-----------------------|---------------------|
| Preconditioning | | |
| Temperature | 110°F +/- 10°F | |
| Humidity | 65% +/- 15% | 10.35 Hrs - 72 Hrs |
| Chamber | | |
| Vacuum | 21" +/- 1.0 inHgA | |
| Humidification/Dwell | 22.5" +/- 1.0 HgA | 60 +30/-0 minutes |
| Gas Inject/Dwell | 10.3" +/- 0.5 Hg rise | 360 +5/-0 minutes |
| Jacket Temperature | 135°F | |
| Aeration | | |
| Temperature | 120°F | 24 Hrs - 72 Hrs Max |

Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Notice to Users and/or Patients in EU:

Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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