OR Camera Cover

Non-Sterile

Aspen Surgical Products Inc.
6945 Southbelt Drive SE
Caledonia, MI 49316
Phone 616-698-7100
Toll-Free 888-364-7004
aspensurgical.com

EMERGO

 Emergo Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands
Made in USA

English

Intended use:

Intended to provide a sterile interface between the operating room camera (non-sterile) and the Surgeon or Nurse (sterile).

Intended User/ Patient Target Groups:

Device is non-patient contacting and is to be used by Healthcare Professionals. Provides a means of manipulating O.R. Camera.

Precaution:

- Verify securement of the camera cover to the adaptor.
- Device is single use only.
- Reuse of device could result in infection/ contamination and/ or device failure which could lead to patient harm.

Sterilization Instructions

- Place device into a kit or sterilization tray.
- Using Ethylene Oxide sterilization method, sterilize according to the following parameters:

| Cycle Parameter | Specification | Duration |
|------------------------|-----------------------|---------------------|
| Preconditioning | | |
| Temperature | 110°F +/- 10°F | |
| Humidity | 65% +/- 15% | 10.35 Hrs - 72 Hrs |
| Chamber | | |
| Vacuum | 21" +/- 1.0 inHgA | |
| Humidification/Dwell | 22.5" +/- 1.0 HgA | 60 +30/-0 minutes |
| Gas Inject/Dwell | 10.3" +/- 0.5 Hg rise | 360 +5/-0 minutes |
| Jacket Temperature | 135°F | |
| Aeration | | |
| Temperature | 120°F | 24 Hrs - 72 Hrs Max |

Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Notice to Users and/or Patients in EU:

Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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AB'deki Kullanıcılara ve/veya Hastalara Yönelik Bildirim: