

## Instructions for use

### Beach Chair Head and Chin Strap 711005

Previously Beatty® reorder # 111309

#### Intended Use:

Intended to support and maintain the position of an anesthetized patient during surgery.

#### Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

#### Precautions:

- For use with SKYTRON® Beach Chair, SCHUREMED™ Beach Chair, STERIS® Beach Chair, STRYKER® Shoulder Chair OZ 7519475 and OZ 7523075.
- Ensure device is properly aligned above eyes and chin.
- Ensure that skin & hair are away from straps when securing device.
- Ensure that the straps are securely fastened to the positioning device.
- Do not overtighten straps on mask.
- This device is single use only.
- Do not sterilize.

#### Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

#### Instructions for Use:

1. Place foam around patient's forehead and chin securely.
2. Secure straps to positioning device.
3. Discard after use.

Skytron® is a registered trademark of Skytron, LLC. ("Skytron"). SchureMed™ is a trademark of Schuerch Corporation ("Schuerch"). Steris® is a registered trademark of Steris Inc. ("Steris"). Stryker® is a registered trademark of Stryker Corporation ("Stryker"). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by Skytron, Schuerch, Steris, or Stryker.



Aspen Surgical Products, Inc.

6945 Southbelt Dr. SE, Caledonia, MI 49316

phone +1 616.698.7100 toll-free +1 888.364.7004

[aspensurgical.com](http://aspensurgical.com)



**R<sub>x</sub>ONLY**