

# Instructions for Use

## SafeClean® Instrument and Channel Cleaning Brushes

### Intended Use Statement

- Intended to remove surgical residue from surgical instruments and endoscopic devices after surgical procedures.

### Instructions for Use

1. Inspect brush between uses and replace if worn, frayed, bent or otherwise damaged.
2. This inspection is the responsibility of the user.
3. Worn bristles are ineffective in cleaning and could damage the instrument.
4. Ensure brush diameter is adequate size for cleaning the device channel.
5. Advance brush through device channel in short strokes to avoid kinking.
6. Do not rotate brush inside channel.
7. To remove gross soil, flush the device with water while using the brush through the channel until the soil is removed.
8. For specific device cleaning instructions, refer to the device manufacturer's IFU.



Aspen Surgical Products, Inc.  
6945 Southbelt Dr. SE  
Caledonia, MI 49316 USA  
Phone 616-698-7100  
Toll-Free 888-364-7004  
[www.aspensurgical.com](http://www.aspensurgical.com)