

Instructions for use

Hoffee® Blue Lasso Suture Retriever 710000, Sterile

Previously Beatty® reorder # 022701

Intended Use:

Intended to aid in the passing of sutures through bone and soft tissue during orthopedic surgery.

Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Precautions:

- When using arthroscopically, visualize the device as it is placed into the joint to avoid unintentional cartilage damage.
- Do not bend the device.
- Do not alter the suture on the device in any way.
- This device is single use only.
- Use caution when opening sterile package, device may roll.
- Do not use product if product is damaged or particulate is found in sterile packaging.

Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Instructions for Use:

1. Used to retrieve suture (surgical equivalent to a needle threader).
2. Place suture through the blue suture loop and pull through joint, bone or soft tissue.
3. Remove device from joint in a smooth motion.
4. Discard after use.



Aspen Surgical Products, Inc.

6945 Southbelt Dr. SE, Caledonia, MI 49316

phone +1 616.698.7100 toll-free +1 888.364.7004

aspensurgical.com

Made in USA



R_x ONLY