

## Instructions for use

### Patient Care Boot Liner 712300

Previously Beatty® reorder # 120201

#### Intended Use:

Intended to support and maintain the position of an anesthetized patient during surgery.

#### Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

#### Precautions:

- For use with Allen® Traction Boot A – 93005 and A – 93000 and Steris® Traction Boote.
- Ensure liner is securely fastened around foot before placing in the positioning device.
- Avoid redundant material that may wrinkle or cause excess pressure.
- This device is single use only.
- Do not sterilize.

#### Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

#### Instructions for Use:

1. Place patient's foot in liner.
2. Secure foot in positioning device ensuring that there is not excess wrinkling.
3. Secure boot to table.
4. Discard after use.

Allen® is a registered trademark of Allen Medical Systems, Inc. ("Allen Medical"). Steris® is a registered trademark of Steris Inc. ("Steris"). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by Allen Medical or Steris.



Aspen Surgical Products, Inc.

6945 Southbelt Dr. SE, Caledonia, MI 49316  
phone +1 616.698.7100 toll-free +1 888.364.7004  
[aspensurgical.com](http://aspensurgical.com)

Made in USA



**R<sub>x</sub> ONLY**