

Instructions for use

Prone Face Pillow 714100

Previously Beatty® reorder # 021555

Intended Use:

Intended to support and maintain the position of an anesthetized patient during surgery.

Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Precautions:

- Ensure that patients face is centered on the pillow.
- Ensure that the prone face pillow is facing up and that the patients face in on the soft foam
- Ensure that intubation tube is secure in the premade holder.
- Frequently check patient neck, head, eyes, nose and mouth to ensure that a safe position is maintained and offload as required per facility protocol.
- This device is single use only.
- Do not sterilize.

Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Instructions for Use:

1. Allow proper expansion of prone pillow prior to use.
2. Carefully place pillow over patients face in the supine position after intubation.
3. Hold pillow in place while patient is being moved to the prone position.
4. Properly align spine in prone position.
5. Check tube placement after patient is in the prone position.
6. Discard after use.



Aspen Surgical Products, Inc.

6945 Southbelt Dr. SE, Caledonia, MI 49316

phone +1 616.698.7100 toll-free +1 888.364.7004

aspensurgical.com

Made in USA



Rx ONLY