

## Instructions for use

### Sedona Arm Positioner Set 714200

Previously Beatty® reorder # 030701

#### Intended Use:

Intended to support and maintain the position of an anesthetized patient during surgery.

#### Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

#### Precautions:

- Secure arms in the cradle to keep in place.
- Single use only.
- Do not sterilize.

#### Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

#### Instructions for Use:

1. Place arm cradle (right and left) on corresponding arm board
2. Make sure cutouts are facing inward.
3. Secure entire arm in the cradle.
4. Discard after use.



Aspen Surgical Products, Inc.

6945 Southbelt Dr. SE, Caledonia, MI 49316

phone +1 616.698.7100 toll-free +1 888.364.7004

[aspensurgical.com](http://aspensurgical.com)

Made in USA



**R<sub>x</sub>ONLY**