# Vessel Loops

(Non-Sterile)

Aspen Surgical Products, Inc.  
6945 Southbelt Drive SE  
Caledonia, MI 49316 USA  
Phone 616-698-7100  
Toll-Free 888-364-7004  
aspensurgical.com

EM REP  
Emergo Europe  
Prinsesgracht 20  
2514 AP The Hague  
The Netherlands

Made in USA

CE  
2797

English

## Intended use:

Vessel loops are intended to be used to occlude, retract, and identify arteries, veins, tendons, and nerves in surgical procedures.

## Intended User/ Patient Target Groups:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

## Sterilization Instructions:

- Place device into a kit or sterilization tray.
- Using Ethylene Oxide sterilization method, sterilize according to the following parameters:

Cycle Parameter	Specification	Duration
<b>Preconditioning</b>		
Temperature	110°F +/- 10°F	
Humidity	65% +/- 15%	10.35 Hrs - 72 Hrs
<b>Chamber</b>		
Vacuum	21" +/- 1.0 inHgA	
Humidification/Dwell	22.5" +/- 1.0 HgA	60 +30/-0 minutes
Gas Inject/Dwell	10.3" +/- 0.5 Hg rise	360 +5/-0 minutes
Jacket Temperature	135°F	
<b>Aeration</b>		
Temperature	120°F	24 Hrs - 72 Hrs Max

## Directions for use:

- After removing loops from package, the surgeon may do as follows:
  - Retract-wrap around vessel. Clamp loop to drape or hang outside wound.
  - Occluding-wrap loop around vessel twice. Clamp adjacent to vessel to secure.
  - Identify-wrap loop around vessel, nerve and ureter or intestine.
- Dispose or recycle general surgical devices per facility protocol.

## Precautions:

Device is single use only. Device is intended for limited contact duration, less than or equal to 8 hours. Reuse of device could result in infection/contamination and/or device failure which could lead to patient harm.

## Notice to Users and/or Patients in EU:

Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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