

## Instructions for use

### MD Traction Rope, Non-Sterile

#### Intended Use:

Intended to support and maintain the position of an anesthetized patient during surgery.

#### Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

#### Precautions:

- For use with any suspension like device that utilizes rope.
- This device is single use only.

#### Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

#### Sterilization Instructions

To ensure SAL of 10<sup>-6</sup>, this product must be sterilized via gamma sterilization method at a sterilization dose range of 25-40kGy.

#### Instructions for Use:

1. Place rope in distraction device according to manufacturer's recommendations.
2. Hook to suspension kit and apply the manufacturer's recommended weight to the device for optimum shoulder distraction.
3. Discard after use.

