

Instructions for use

Seattle Brachy Bed 715100

Previously Beatty® reorder # 081601

Intended Use:

Intended to support and maintain the position of an anesthetized patient during surgery.

Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Precautions:

- Additional pads or straps may be necessary for certain procedures to help prevent patient from slipping while positioning.
- Single use only.
- Do not sterilize.

Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Instructions for Use:

1. Place mattress on OR table.
2. Position patient on the mattress.
3. Discard after use.



Aspen Surgical Products, Inc.

6945 Southbelt Dr. SE, Caledonia, MI 49316

phone +1 616.698.7100 toll-free +1 888.364.7004

aspensurgical.com



Rx ONLY