

Instructions for use

Thigh Holder Pad 713500

Previously Beatty® reorder # 121601

Intended Use:

Intended to support and maintain the position of an anesthetized patient during surgery.

Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Precautions:

- For use with Smith & Nephew® FERKEL™ Thigh System 72201812.
- Ensure thigh is completely in holder after placing in the positioner.
- This device is single use only.
- Do not sterilize.

Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Instructions for Use:

1. Place thigh in holder on the pad.
2. Discard after use.

Smith & Nephew® and FERKEL™ are registered trademarks or trademarks of Smith & Nephew PLC (Smith & Nephew). Aspen Surgical Products, Inc. is not affiliated with, endorsed by, or sponsored by Smith & Nephew.



Aspen Surgical Products, Inc.

6945 Southbelt Dr. SE, Caledonia, MI 49316

phone +1 616.698.7100 toll-free +1 888.364.7004

aspensurgical.com



Rx ONLY