

Instructions for use

Instruments Guards

Includes all products except for Web Guards #09-9-001 through 09-9-007

Intended Use:

The Instrument Guards are intended to protect surgical instruments through handling and sterilization.

Intended User:

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Instructions for Use:

- Place guard over clean instrument.
- Guards maybe left on during handling, storage, and sterilization by steam or ethylene oxide.
- Flash steam sterilize for 4 minutes at 132° C (270° F).
- Conventional steam sterilize for 30 minutes at 121° C (250° F).

Disposal:

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.



Aspen Surgical Products, Inc.

6945 Southbelt Dr. SE, Caledonia, MI 49316

phone +1 616.698.7100 toll-free +1 888.364.7004

aspensurgical.com

Made in USA



Rx ONLY