INSTRUCTIONS FOR USE

Dr. Fog®

aspensurgical.

6g Anti-Fog Solution, Sponge

Aspen Surgical Products, Inc.
6945 Southbelt Drive SE
Caledonia, MI 49316 USA
Phone +1 616-698-7100
Toll-Free 888-364-7004
aspensurgical.com

EMERGEO
Emergo Europe
Westervoortsewijk 60
6827 AT Arnhem
The Netherlands

CE
2797

For Country of Origin See Product Label

English

Intended Use

Intended as an anti-fog agent for endoscopic camera lenses.

Intended User/ Patient Target Groups

Intended to be used by Healthcare Professionals on patients requiring medical procedures.

Instructions for Use:

- Remove paper backing from sponge and place sponge onto sterile drape.
- Remove cap and apply drops of Anti-Fog solution to sponge.
- Touch end of scope to sponge, blot. (do not wipe) with sterile sponge.
- A singular anti-fog device is intended to prevent fogging when used and reapplied every 15 minutes up to 1 hour.
- Do not use product if product is damaged.

Precaution

Avoid contact with eyes and prolonged contact with skin.

Warning

Reuse of device could result in infection/contamination and/or device failure which could lead to patient harm.

Disposal

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Notice to Users and/or Patients in EU

Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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AB'deki Kullanıcılar ve/veya Hastalara Yönelik Bildirim
Emergi Consulting (UK) Limited