INSTRUCTIONS FOR USE

Light Handle Adaptors

Non-Sterile

Aspen Surgical Products Inc.
6945 Southbelt Drive SE
Caledonia, MI 49316
Phone +1 616-698-7100
Toll-Free 888-364-7004
aspensurgical.com

EMERGO
EMERGO Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

For Country of Origin See Product Label

English

Intended Use

Light Handle adaptors are used as an interface between non-sterile surgical light and sterile Aspen Surgical's disposable light handles.

Intended User/ Patient Target Groups

Device is non-patient contacting and is to be used by Healthcare Professionals.

Installation

Refer to appropriate Installation Instructions for affixation to surgical light.

Instructions for Use:

- Disinfecting of Light handle adaptors should be performed per facility protocol.
- Verify securement of the light handle cover to the adaptor.
- Do not use if product appears visually damaged or degraded.
- Provides a means of manipulating O.R. Lights.
- Both rigid handle adaptors and flexible handle adaptors are mechanically secured to the O.R. light and are intended to remain installed permanently.
- Adaptors will be delivered non-sterile, it is intended that users will not sterilize the adaptors.

Disposal

Dispose or recycle general surgical devices per facility protocol and all applicable federal, state, regional, and/or local laws and regulations.

Notice to Users and/or Patients in EU

Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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AB'deki Kullanıcılara ve/veya Hastalara Yönelik Bildirim
Emergo Consulting (UK) Limited